


## 3M™ Tegaderm™ Transparent Film Roll

*All the benefits of Tegaderm™ Transparent Film Dressings in a non-sterile roll*

# Tegaderm™ Transparent Film Roll

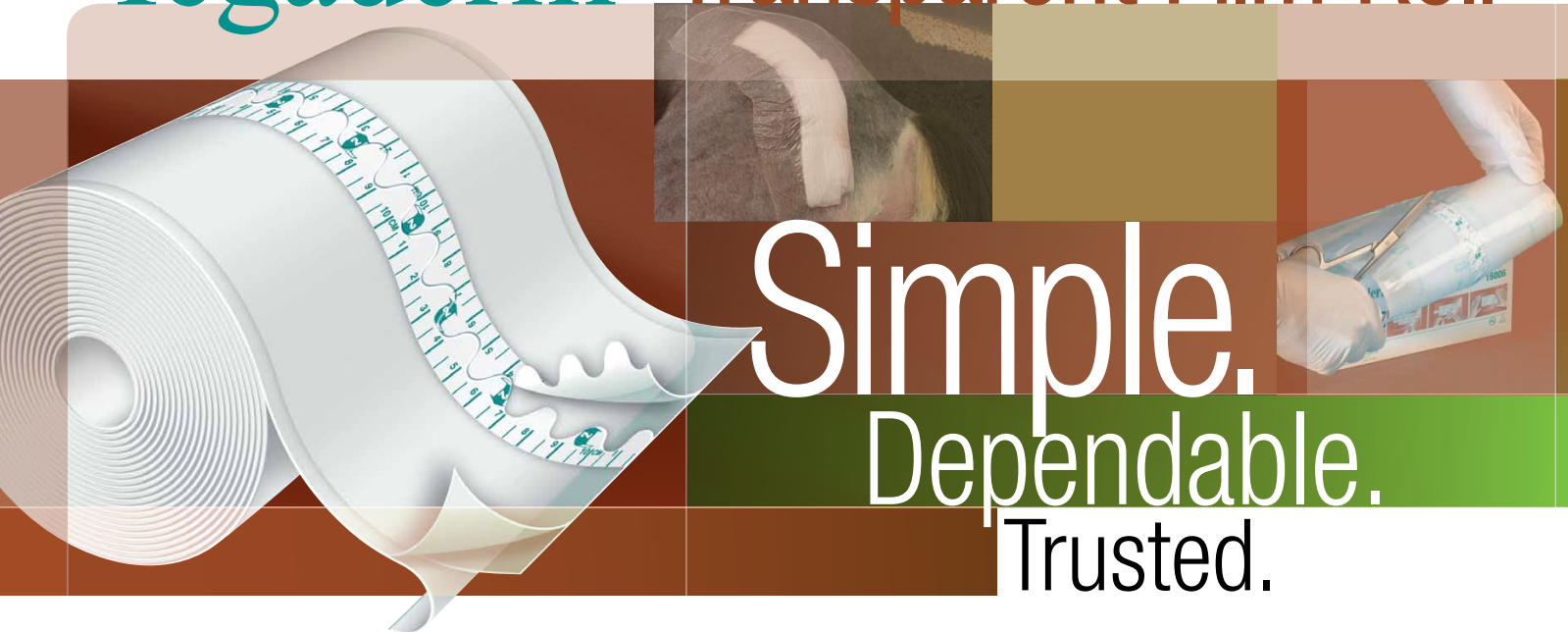

Simple.  
Dependable.  
Trusted.

### **A non-sterile version of Tegaderm™ Transparent Film Dressings**

Tegaderm™ Roll provides maximum versatility and is the affordable choice for skin protection and waterproof cover dressings.

- The roll version allows for maximum versatility.
- The perfect solution whenever a waterproof cover dressing is the appropriate choice.



# 3M™ Tegaderm™ Transparent Film Roll

## Unique Features



- Thin film is easily cut to desired shape or size. If needed, corners can be shaped and rounded.
- Conformable – Fits body contours, permits free movement without stress on skin.
- Waterproof, bacterial *and viral*\* barrier, helps to prevent external contamination of the site, as well as helps to protect the patient and healthcare workers.
- Easy stretch and release removal.

\**In vitro* testing shows that the transparent film of Tegaderm™ and Tegaderm™ HP dressings provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

## Application Instructions



**Step 1**

Cut to desired length and round corners or cut to shape for given application.



**Step 2**

Remove the backliner. Hold dressing by side tabs. Center over dressing or area requiring fixation or protection and apply without stretching.



**Step 3**

Gently smooth down the dressing, helping to seal the film. Remove the top liner.

## Removal Instructions



**Step 1**

Grasp one edge of the dressing and gently pull straight out to stretch it and release adhesion, carefully holding onto skin with other hand.



**Step 2**

If necessary, press a small piece of tape onto a corner to start an edge of the dressing. By stretching the dressing, the material expands and releases until the dressing is removed.

## Ordering Information

Catalog No. NDC No.	Size	Rolls/ Box	Boxes/ Case
3M™ Tegaderm™ Transparent Film Roll (Non-Sterile)			
<b>16004</b>	4 in x 11 yd 10 cm x 10 m	1	4

For additional information, contact your 3M Animal Care representative, or call the 3M Customer Service Helpline at **1-800-848-0829**.



**Skin and Wound Care Division**  
**3M Animal Care Products**  
 3M Center, Building 275-5W-05  
 St. Paul, MN 55144-1000  
 U.S.A.  
 1 800 848-0829  
[www.3M.com/animalcare](http://www.3M.com/animalcare)

**3M Canada Company**  
 Post Office Box 5757  
 London, Ontario N6A 4T1  
 Canada  
 1 800 410-6880

3M and Tegaderm are trademarks of 3M.  
 Used under license in Canada.  
 Please recycle. Printed in U.S.A.  
 © 3M 2011. All rights reserved.  
 70-2009-2699-9